

MANAGE YOUR ERECTION PROBLEMS with POS-T-VAC®

INTRODUCTION

By the time you read this you may have tried various tablets, injections or other treatments with limited or no success. Erection problems result from many causes including diabetes, prostate and other surgery, a wide range of medications and just plain getting older. Severity can vary from “going soft” during intercourse to a complete inability to achieve an erection. Inadequate filling of the penis with blood or an incompetent “holding” mechanism (venous leakage), sometimes a combination of both, needs to be addressed. Oral and injectable medications will improve the flow of blood into the penis and this is often enough to obtain and maintain the erection. Sometimes additional assistance is needed to gain the desired rigidity and then to hold the blood in the penis during intercourse.

VACUUM ERECTION DEVICES (VED's) provide a simple mechanical means of producing the desired rigidity for penetration and maintaining the erection for satisfactory sexual intercourse. They are effective in almost every situation. Basically, all VED's consist of a clear penile tube, a vacuum pump and a series of elastic support rings plus accessories. The clear plastic tube of the pump unit is placed over the penis and against the body. When the pump is activated a vacuum is created around the penis encouraging inflow of blood from the body to produce the required rigidity. A support ring pre-loaded onto the tube (see pictures) is then transferred onto the base of the penis compressing the large veins on the outside of the penis thereby slowing the return of blood to the body when the vacuum is removed. This ring may be left on for up to 30 minutes.

HOW IS THE ERECTION DIFFERENT?

Erectile tissue runs the length of the penis and back inside the body to give internal support to the penis for directional control and “thrustability”. As vacuum devices can only create an erection forward from the support ring at the base of the penis this control is dependent on the strength of erection you can achieve naturally. The worst-case scenario in a man unable to obtain any sort of erection is that the penis, even though hard, will tend to hang down and needs to be guided into place manually. Internal support will depend on the existing level of erection you can obtain without assistance or with the aid of oral or injectable medications.

An Israeli study found that men who were not entirely happy with either Viagra® or vacuum therapy alone reported overall increased satisfaction when treated with a combination of both. If necessary you may wish to discuss this possibility with your treating doctor.



MVP-700 Premium Manual Pump
Lifetime Warranty
Easy Load band loader included



IVP-600 Standard Manual Pump
12 Month Warranty
Easy Load band loader included

Please contact Michael Swan, (03) 9550 1819 to discuss your potential for success with POS-T-VAC®

“EASY LOAD” Ring Loader



“EASY LOAD” Ring Loader, included with both vacuum devices is particularly useful for men with dexterity problems and those needing to use smaller sized bands to maintain their erection.

REJOYN



Rejoyn is a simple support ring system that can be used alone for mild erection problems (“going flat”) or in conjunction with oral or injectable medications to provide or maintain extra hardness.

The concept is to slow the venous return of blood to the body thus maintaining the erection. This system is of little value if an initial erection of about 90% cannot be firstly achieved.

ORDER FORM

Phone us Toll Free 1800 032 404 to confirm prices.

Credit card purchases may be made by telephone or mail your payment with this order form to:

Endovision Pty Ltd
21 Aristoc Rd, Glen Waverley, 3150, Victoria

Name:

Street:.....

Suburb: P/C

State: Tel: (.....)

Please supply: MVP-700 (\$663), IVP-600 (\$545)

REJOYN (\$66) Amount: \$

Prices include GST and Express Post/Post delivery.

Method of payment: (Please do not send cash)

- | | |
|-------------------------------------|--|
| <input type="checkbox"/> Mastercard | <input type="checkbox"/> Personal Cheque |
| <input type="checkbox"/> VISA | <input type="checkbox"/> Money Order |
| | <input type="checkbox"/> Bank Cheque |

Card Number:

Expiry Date:/.....

Name on Card:

Signature: